Produced By HuVeSearch BVBA/SPRL
Europalaan, 11 - 3900 PELT - BELGIUM

Hippex CM, Centrifugation Media for Equine Sperm


Operating instructions

HIPPEX CENTRIFUGATIEMEDIUM (HCM) consists of HIPPEX enriched with egg yolk (LDL).

The general guidelines about sperm freezing like for example centrifugation time and speed, target sperm concentration, straws filling, freezing methods and protocols are considered as known.

1. Take the frozen media out of the freeze 24 hours in advance of use and place it in the fridge or take it out of the freezer 2 hours before use and place it in water bath at 30°C.
2. Dilute the collected sperm with the centrifugation media HIPPEX with a factor 1:1.
3. Centrifuge the mix and take out the supernatant with a disposable pipet.
4. Resuspend the pellet (concentrated sperm) to the targeted final concentration in the HIPPEX CRYO media to obtain the desired concentration. Place the preparation on a shaking table or a rolling agitator for 15 minutes in order to optimize the mixing and stabilize to the equilibrium.
5. Fill the straws with the prepared diluted semen and proceed to the freezing steps with the selected protocol.

English

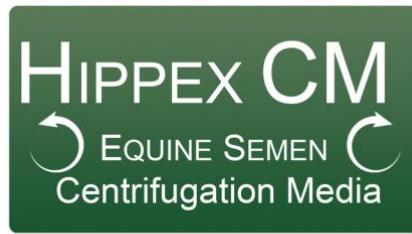Produced By HuVeSearch BVBA/SPRL
Europalaan, 11 - 3900 PELT - BELGIUM

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